

flexGrip®

Sterile, single use patch for stabilization of catheters, tubes and cables

The device is used in combination with a compatible catheter, tube or cable. The device consists of an adhesive dressing and a clamp to reduce catheter/tube/cable movement. It is to be used specifically for therapeutic purposes and intended to be used for treatment of disease. The device is used to stabilize a catheter or tube or cable. It is attached to the skin for a maximum of 7 days.

Indications for use:

- The device is a stabilization device for compatible catheters, tubes and cables.
- The device is intended to be used by trained medical professionals in a hospital or clinical setting.

Contraindications:

- Known tape or adhesive allergies.
- Do not use if package is damaged.
- Do not use if the device is damaged.
- Key points to read before use:

- Product has an expiry date.
- Protect the device from damage.
- Secure the compatible catheter, tubing or cable in the Bedside Redal.
- Apply the device to the skin.
- In case of loss or difference in adhesion to the skin, the device should be removed and replaced.
- Read all instructions, warnings, precautions and instructions prior to use. Failure to do so may result in severe patient injury.
- Do not use if the device is damaged.
- Discard if package is damaged.

The extra strip can be used to temporarily secure the catheter during dressing change.

Discard after use in appropriate containers for potentially infectious material.

CE 1639 Rx Only

Incidents should be reported to the manufacturer.

flexGrip®

Sterile ühekordne kateetrite, voolikute ja kaablite kinnitamiseks

Kavandust kasutuse: Toode kasutatakse koos ühildava kateetri, voolikuga või kaabliga. Toode on ühekorne ja ei sisalda keelatud läänevalmistatust, mis viibendab kateetri liikumist.

Toode on mõeldud kasutuseni koljutatud läänevalmiste protsessidega.

Ettevaatust: Käsiteerite.

Toode on mõeldud kateteeriteks, tubede ja kaabliteks.

Vastunulikustus: Eriti vähe.

Käsiteeriteks: Eriti vähe

bedal flex

Reference	Description	Size range	Device			
UN-FLEX-XS	bedal flex, size XS	2.7-5 Fr				http://lead.me/bbQGJw

bedal International

AR	BG	CS	DA	DE	EL	ES	ET	FA	FI	FR
BEDAL International Agoralan a bie-3590 Diepenbeek Belgium E info@bedal.be										
T+32(0)11/28.69.02 - www.flex-grip.com										
Assembled in Poland, EU	مُسَمَّةٌ بِإِنْجَامِ الْأَعْوَادِ	Собрано в Польше, ЕС	Sestaveno v Polsku, EU	Samlet i Polen, EU	Zusammengebaut in Polen, EU	Συγκρατήση στην Πολωνία, ΕΕ	Ensamblado en Polonia, UE	Kokku pandud Poolas, EL-IS	وصلت هذه المعايير إلى	Kasattu Puolassa, EU
Reference	ترخيص	Prerazna	Bezvred	Referencia	Vile	Συγκράτηση στην Πολωνία, ΕΕ	Ensamblado en Polonia, UE	Viite	Assemblé en Pologne, UE	
Description	ووصف	Описаниe	Beskrivelse	Referencia	Vile	Συγκράτηση στην Πολωνία, ΕΕ	Ensamblado en Polonia, UE	Viite	Assemblé en Pologne, UE	
Size range	مدى الحجم	Диапазон размеров	Rozsah velikosti	Størrelsesområde	Größenbereich	Εύρος μεγέθους	Suuruus	مدى الحجم	Viite	Assemblé en Pologne, UE
Device	آلات طبية	Медицинское изделие	Zdravotnický prostředek	Medicinsk anordning	Medizinprodukt	Ιατροτεχνολογικό προϊόν	Prodotto sanitario	Meditsiiniline seade	Laite	Assemblé en Pologne, UE
MD	لا تستخدم إذا كان المحتوى مضرًا	Не подходит, покуда опасности не устранены	Nepoužívejte, pokud je poškozený	Nicht verwenden, wenn Verpackung beschädigt	Не пригоден для использования, если упаковка повреждена	Δεν μπορείται να χρησιμεύεται αν η συσκευασία δέσμη υγείας	Är ganska svårt att använda om förpackningen är skadad	كما يشترى، لا يمكن استخدامه	Ei saa käyttää, jos pakkaus on vaurioitunut	Assemblé en Pologne, UE
Do not use if package is damaged	لا تستخدم إذا كانت المغصبة مكسورة	Не подходит, покуда опасности не устранены	Nepoužívejte, pokud je poškozený	Nicht verwenden, wenn Verpackung beschädigt	Не пригоден для использования, если упаковка повреждена	Δεν μπορείται να χρησιμεύεται αν η συσκευασία δέσμη υγείας	Är ganska svårt att använda om förpackningen är skadad	كما يشترى، لا يمكن استخدامه	Ei saa käyttää, jos pakkaus on vaurioitunut	Assemblé en Pologne, UE
Date of manufacture	تاريخ الصنع	Дата производства	Datum výroby	Fremstillingdato	Herstellungsdatum	Ημερομηνία κατασκευής	Tootmiskuupäivä	تاریخ تولید	Viite	Assemblé en Pologne, UE
STERILE R	مُستериلاً بالأشعة	Стерилизовано радиацией	Sterilizován pomocí rádiových zářiv	Steriliseret ved anvendelse af bestrålning	Durch bestrahlung sterilisiert	Αποτροπής από τη χρήση ακτινοβολίας	Sterilisiertert kriamtisiega	مُستериلاً بالأشعة	Kuvaus	Assemblé en Pologne, UE
LOT	رقم الشفافية	Номер на партида	Číslo sárže	Varenúmer	Lotnummer	Αριθμός παραγωγής	Partili numero	شماره لام	Viite	Assemblé en Pologne, UE
Consult instructions for use	الخطوات الاستخدام	Важные указания по употреблению	Pozrite si instrukcie k použitiu	Brugsanvisninger	Gebrauchsweisung	Συσκευεπίδειος χρήσης	Consultar las instrucciones de uso	مُستериلاً بالأشعة	Viite	Assemblé en Pologne, UE
Do not re-use	لا إعادة	Не использовать повторно	Nepoužívejte znova	Ger ingen genanvendelse	Nicht wieder verwenden	Δεν μπορείται να χρησιμεύεται πάλι	Är ganska svårt att använda igen	كما يشترى، لا يمكن استخدامه	Ei saa käyttää uudelleen	Assemblé en Pologne, UE
Use-by date	التاريخ المأمول	Срок годности	Datum expirace	Anwendungsdatum	Verwendung bis zum Datum	Ημερομηνία λήξης	Fechtercaducidad	تاریخ انتهاء صلاحية	Viimeinen käyttöpäivä	Assemblé en Pologne, UE
Manufacturer	الشركة المصنعة	Производитель	Výrobce	Producent	Hersteller	Κατασκευαστής	Tootja	مُصنِّع	Viite	Assemblé en Pologne, UE
Latex free	خلية من اللاتكس	Не содержит латекса	Latex zderma	Latexfri	Frei von latex	Δεν περιέχει λάτεξ	Libre de latex	بدون لاتكس	Lataksiton	Assemblé en Pologne, UE
Temperature limit	حد درجة الحرارة	Температурная граница	Teplota omezit	Temperaturgrænse	Temperaturgrenze	Όρος θερμοκρασίας	Temperaturupplirang	محدود درجة حرارة	Määrätietoinen	Assemblé en Pologne, UE
REF	رقم الفيلم	Код изделия	Katalog číslo	Katalognummer	Katalognummer	Αριθμός καταλόγου	Kataloginumero	مُستериلاً بالأشعة	Viite	Assemblé en Pologne, UE
Caution	تحذير	Предупреждение	Pozor	Forsiktig	Vorsicht	Βραχιόνων προστασία	Precaución	تحذير	Viite	Assemblé en Pologne, UE
Scan QR code to view instruction video. Video only shows IFU partly.	ابحث عن الرمز الـQR لمشاهدة الفيديو التعليمي. الفيديو يوضح جزءاً من المنشور.	Scan QR-koden för att se instruktionsvideo. Video visar delvis IFU.	Scan QR-koden für die Anleitungsvideo. Das Video zeigt nur teilweise das IFU.	Scan QR-koden für die Anleitungsvideo. Das Video zeigt nur teilweise das IFU.	Scan QR-koden für die Anleitungsvideo. Das Video zeigt nur teilweise das IFU.	Επανεπαنے	Assemblé en Pologne, UE			
MD	لا تستخدم إذا كان المحتوى مضرًا	Не подходит, покуда опасности не устранены	Nepoužívejte, pokud je poškozený	Nicht verwenden, wenn Verpackung beschädigt	Не пригоден для использования, если упаковка повреждена	Δεν μπορείται να χρησιμεύεται αν η συσκευασία δέσμη υγείας	Är ganska svårt att använda om förpackningen är skadad	كما يشترى، لا يمكن استخدامه	Ei saa käyttää, jos pakkaus on vaurioitunut	Assemblé en Pologne, UE
Do not use if package is damaged	لا تستخدم إذا كانت المغصبة مكسورة	Не подходит, покуда опасности не устранены	Nepoužívejte, pokud je poškozený	Nicht verwenden, wenn Verpackung beschädigt	Не пригоден для использования, если упаковка повреждена	Δεν μπορείται να χρησιμεύεται αν η συσκευασία δέσμη υγείας	Är ganska svårt att använda om förpackningen är skadad	كما يشترى، لا يمكن استخدامه	Ei saa käyttää, jos pakkaus on vaurioitunut	Assemblé en Pologne, UE
Date of manufacture	تاريخ الصنع	Дата производства	Datum výroby	Fremstillingdato	Herstellungsdatum	Ημερομηνία κατασκευής	Tootmiskuupäivä	تاریخ تولید	Viite	Assemblé en Pologne, UE
STERILE R	مُستериلاً بالأشعة	Стерилизовано радиацией	Sterilizován pomocí rádiových zářiv	Steriliseret ved anvendelse af bestrålning	Durch bestrahlung sterilisiert	Αποτροπής από τη χρήση ακτινοβολίας	Sterilisiertert kriamtisiega	مُستериلاً بالأشعة	Kuvaus	Assemblé en Pologne, UE
LOT	رقم التسلسلي	Номер на партида	Číslo sárže	Varenúmer	Lotnummer	Αριθμός παραγωγής	Jäljige numer	شماره لام	Viite	Assemblé en Pologne, UE
Consult instructions for use	الخطوات الاستخدام	Важные указания по употреблению	Pozrite si instrukcie k použitiu	Brugsanvisninger	Gebrauchsweisung	Συσκευεπίδειος χρήσης	Consultar las instrucciones de uso	مُستериلاً بالأشعة	Viite	Assemblé en Pologne, UE
Do not re-use	لا إعادة	Не использовать повторно	Nepoužívejte znova	Ger ingen genanvendelse	Nicht wieder verwenden	Δεν μπορείται να χρησιμεύεται πάλι	Är ganska svårt att använda igen	كما يشترى، لا يمكن استخدامه	Ei saa käyttää uudelleen	Assemblé en Pologne, UE
Use-by date	التاريخ المأمول	Срок годности	Datum expirace	Anwendungsdatum	Verwendung bis zum Datum	Ημερομηνία λήξης	Fechtercaducidad	تاریخ انتهاء صلاحية	Viimeinen käyttöpäivä	Assemblé en Pologne, UE
Manufacturer	الشركة المصنعة	Производитель	Výrobce	Producent	Hersteller	Κατασκευαστής	Tootja	مُصنِّع	Viite	Assemblé en Pologne, UE
Latex free	خلية من اللاتكس	Не содержит латекса	Latex zderma	Latexfri	Frei von latex	Δεν περιέχει λάτεξ	Libre de latex	بدون لاتكس	Lataksiton	Assemblé en Pologne, UE
Temperature limit	حد درجة الحرارة	Температурная граница	Teplota omezit	Temperaturgrænse	Temperaturgrenze	Όρος θερμοκρασίας	Temperaturupplirang	محدود درجة حرارة	Määrätietoinen	Assemblé en Pologne, UE
REF	رقم الفيلم	Код изделия	Katalog číslo	Katalognummer	Katalognummer	Αριθμός καταλόγου	Kataloginumero	مُستериلاً بالأشعة	Viite	Assemblé en Pologne, UE
Caution	تحذير	Предупреждение	Pozor	Forsiktig	Vorsicht	Βραχιόνων προστασία	Precaución	تحذير	Viite	Assemblé en Pologne, UE

flexGrip® universal

Reference	Description	Size range	Device			

<tbl_r cells="7" ix="2" maxcspan="1" maxrspan="1